
MANUAL –

Botect Nose-mouth facial protection mask

Model M0110030

Version: 0 – Most recent manual version 07/04/2020

Manufacturer Data

Manufacturer: Valigeria Roncato Spa

Address: Via Pioga, 91, 35011 Campodarsego (PD), Italy

VAT: 00332950286

Registered e-mail: valigeriaroncato@legalmail.it

Tel.: 049 9290555

E-mail: infoline@roncato.com

Website: www.roncato.com

Summary

Manufacturer Data	1
AUTHORIZED ASSISTANCE	1
Product assistance can only be performed by authorized and qualified personnel from the manufacturer.	1
INTRODUCTION	1
INTENDED USE	2
WARRANTY	2
Use of the warranty:.....	3
Parts NOT covered by warranty:	3
PRODUCT DESCRIPTION	3
TECHNICAL DATA	3
Unauthorized Uses	3
Product Limitations	3
RESIDUAL RISKS	4
INDICATIONS FOR USE:.....	4
CLEANING AND MAINTENANCE	4
STORAGE.....	4
WARNING	5

AUTHORIZED ASSISTANCE

Product assistance can only be performed by authorized and qualified personnel from the manufacturer.

INTRODUCTION

This manual contains instructions for the use of the washable nose-mouth facial protection mask of the BOTECT line model M0110030 (hereinafter "Product").

This manual is intended for the user to explain the use and storage of the Product and the technical life after its production and sale. This manual contains proprietary information and cannot be even partially provided to third parties for any use and in any form without the manufacturer's prior written consent.

Information on the MANUFACTURING COMPANY (hereinafter, "Manufacturer"):

The Valigeria Roncato Spa manufacturer declares that the information contained in this manual is congruent with the technical and safety specifications of the product to which the manual refers. The Manufacturer does not recognize any documentation that has not been produced, released or distributed by itself or by its authorized representatives.

This manual, along with the entire technical file, will be kept by the manufacturer for the period required by law.

INTENDED USE

The product **is neither a medical device (MD) nor personal protective equipment (PPE).**

The purpose of the Product is not to protect the user's respiratory tract (for example against solid, liquid aerosols or against irritants or dangerous, toxic or radiotoxic gases) and cannot be used in the health sector or in the workplace where the use of PPE is expressly mandatory.

The Product is an individual face, nose and mouth protection device with interchangeable and adjustable elastic filters, and is intended for general precautionary use. It can be effective against drops and non-oily particles, oily particles, dust, wind and the spray of saliva and mucus.

WARRANTY

The warranty rules, listed in full in the purchase contract, are valid only if the Product is used in accordance with the conditions and terms of use. The manufacturer guarantees that the Product does not cause damage or cause additional risks for users who use it according to the intended use envisaged by the Manufacturer.

Any repair or modification made to the Product by the user or by unauthorized companies voids the warranty. The warranty does not extend to damage caused by inexperience or negligence in the use of the product.

The Product warranty is subject to the following conditions:

1. The warranty applies to defects in the conformity of the Product and is valid for a period of twenty-four (24) months from delivery: without prejudice to the foregoing terms, the lack of conformity of the Product must be reported to the Manufacturer, under penalty of voiding the warranty, within:
 - two (2) months from the date of the discovery of the defect, in the case of a consumer;
 - one (1) month from the date of the discovery of the defect, in the case of a non-consumer buyer.
2. The warranty **IS NOT APPLICABLE** and does not cover consumer defects, deterioration, contamination, or improper or incorrect use of the Product or parts thereof;
3. The Manufacturer reserves the right to decide whether to replace or repair the Product that lacks conformity and only after careful inspection and verification of incorrect construction. The rights of the Consumer pursuant to the Consumer Code are reserved (art. 130 Consumer Code).
4. Transport and / or shipping costs are the responsibility of the buyer in cases of incorrect use of the warranty terms.
5. During the warranty period, the replaced components become the property of the Manufacturer.
6. The warranty operates solely in favor of the original purchaser who has respected the instructions for normal use contained in the manual. The warranty lapses and ceases when the original owner cedes ownership of the product, or when changes have been made to it.
7. The warranty does not include damage resulting from excessive stress caused by, e.g., use of the product after the discovery of an anomaly, the use of inadequate operating methods or the failure to observe instructions for use.

8. The manufacturer assumes no responsibility for any difficulties that may arise in the resale or use abroad due to the provisions in force in the country in which the product was sold.

Use of the warranty:

For complete warranty conditions, consult the website or contact the manufacturer, in any case, if use of the warranty is deemed necessary, please indicate the following data:

- 1 Purchase date (with the purchase document)
- 2 Detailed description of the problem.

Parts NOT covered by warranty:

- 1 The depletion of filters is NOT covered by the warranty;
- 2 Incorrect use of the product or parts of it (e.g. elastic bands) is NOT covered by the warranty;
- 3 Improper use, inadequate maintenance or cleaning of the Product and its parts are NOT covered by the warranty.

PRODUCT DESCRIPTION

The Product is a reusable face protection mask made in plastic with a TNT filtering section that prevents saliva and mucus from being projected against third parties and / or things.

It can be effective against: drops and non-oily particles, oily particles, dust, wind and sprays of saliva and mucus.

THE Product is **NOT** exempt from the provisions of current legislation, any use of this item other than those specified by the standards is to be considered improper and inadequate for your own safety and potentially dangerous to other people.

TECHNICAL DATA

Dimensions (mm)		L 13 x P 13 x H 7
Weight (gr)		51,6
Composition:	Facial protection	100% medical PVC
	Filter "A"	TNT 100% polypropylene
	Filter "B"	TNT 100% polyester
	Filter Locking Cap	100% polypropylene (PP)
	Tape Binding	Elastic material
	Elastic blocking rings	100% polypropylene (PP)
Size		Single, adult

Unauthorized Uses

The following uses are absolutely forbidden: the use of the Product as Personal Protection Equipment or Medical Device, use of the product by persons who have not read the manual carefully or understood the indications for use, use by persons allergic to the substances the product is composed of, use by children for recreational activities, the application of the mask on injured skin or wounds, the use of the product on parts of the body other than the face.

Product Limitations

- The Product can be used until the structure is compromised (e.g. cuts, cracks...)
- The filter section must be replaced regularly, and when breathing becomes difficult.

RESIDUAL RISKS

It is absolutely forbidden to make any changes to the product. Any damage resulting from the use of the product modified incorrectly relieves the Manufacturer of any responsibility. Preserve this manual, as it is necessary for a correct and safe use of the Product. If the structure of the Product shows deformations following an accidental impact, which could include rubbing against an abrasive material, such as to make it potentially dangerous and / or unusable, it is necessary to contact authorized assistance and follow their instructions as reported on the Manufacturer's website.

INDICATIONS FOR USE:

- Check that the mask is clean and the filter section usable. If not, carry out the operations indicated in the "Cleaning and Maintenance" paragraph to replace the filters.
- Sanitize your hands by washing them thoroughly with soap and water.
- Remove the mask from the packaging (in the case of first use).
- Follow the Product instructions included for its use.
- Install the filter section with extreme care and hygiene, taking particular care not to contaminate the packaging of the unused filters.
- Position the filtering section in the quantities and in the order established for maximum performance and comfort.
- Sanitize the Product before and after each use.
- Replace the filter section after each use and verify it is functioning correctly.
- Follow the instructions for replacing the filters.

CLEANING AND MAINTENANCE

After each use, it is necessary to sanitize the Product in all its parts, both internally and externally, by gently wiping it with a damp cloth soaked with an alcohol-based detergent or generic disinfectants. Replace the TNT filtering section after each use of the Product with a new one.

Replacing the filter section:

- Remove the filter locking cap (pos. 3)
- Remove the TNT filter section (pos. 2A and pos. 2B) and dispose of it according to the provisions of your municipality of residence.
- Sanitize the face protection and the cap both internally and externally.
- Place the new filter section in the position and order shown (2A and 2B) taking care not to contaminate it.
- Reposition the filter locking cap on top of the filter section in the dedicated slot.
- Check that the filter locking cap is firmly inserted in its slot and that it holds the filter section in place.

STORAGE

Before storing the Product, check that it is clean and sanitized.

Store the Product in a cool, indoor environment, away from atmospheric agents and heat sources.

- Keep the Product away from open flames and direct sunlight.
- It is advisable to place the Product inside a container capable of preserving its characteristics.
- Store the provided interchangeable filters carefully, avoiding possible direct or indirect contamination.

DISPOSAL

The materials of the Product do not require special disposal procedures. Refer to local regulations

for the disposal of undifferentiated waste (see table of technical data for the composition of the materials). Do not dispose of or abandon the Product in the environment for any reason.

WARNING

The product is a nose-mouth facial protection mask, it is NOT a medical surgical device, it is NOT Personal Protection Equipment. The Product does not protect against harmful gases, pathogens and / or viruses. DO NOT use the Product in contaminated, hazardous to health or low oxygen environments. Failure to replace the filter section or contamination of it, hygiene and cleaning of the Product are the responsibility of the user. The Manufacturer is not liable for incorrect use, contamination or poor hygiene of the Product and its parts.